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1Foreword	4
1.1General.....	4
1.2Design	4
1.3Finally	4
2..Safety	4
2.1Preliminary notes.....	4
2.2Safety belt	5
2.3Emergency stop	5
2.4Access ramp.....	5
2.5Product liability	5
3Indications and contra indications	5
3.1Indications for cardiovascular training.....	5
3.2Absolute contra-indications for cardiovascular training.....	5
3.3Relative contra-indications for cardiovascular training.....	5
4Installation	6
4.1Connection	6
4.2Connection mains cable	6
4.3Switching on and self test	6
4.4Checking walking surface	6
4.5System adjustments (figure 3)	6
4.6Installation	7
4.7Moving and transport	7
4.8Electromagnetic interference	7
4.9Connecting Peripherals.....	7
5Operation	8
5.1Operation units / parts	8
5.2Control panel	8
5.3Execution of training protocols (figure 3)	9
5.3.1Manual mode	9
5.3.2Cardio mode	10
5.3.3Protocol training with time and speed	10
5.3.4Protocol training with time and heart-rate.....	10
5.3.5Quick start.....	11
5.3.6Heart-rate.....	11
5.3.7Stopping a program.....	11
6Maintenance	12
6.1Technical maintenance	12
6.2Alignment of the belt	12
6.3Maintenance (by the user)	12
6.4End of life Mill	13
7Trouble shooting	13
7.1Display does not light up	13
7.2Error-code	13
8Technical specifications	13
Figuren – Pictures – Abbildungen – Figure – Figura	15

1 Foreword

1.1 General

We are very pleased that you have decided to purchase the **M 100** treadmill. This treadmill incorporates the latest technology. A great deal of care has been devoted to its reliability, accuracy, safety and durability.

The **M 100** is a training unit and can be used in range from fitness- till rehabilitation purposes. It is no diagnostic machine. If you need to make a diagnosis, other equipment is required (for example an ECG unit, sphygmomanometer, electrocardioscope and pulsimeter).

When exercising with persons with low cardiac condition, the presence of a qualified person (specialist or therapist) is required to interpret the given values and physical condition of the person.

Using the **M 100** with heartbeat control can only be used for training purposes. Please make sure that these operating instructions are always accessible to personnel who will be involved in the use of the treadmill.

The following symbols are used in this manual

courier font is used for text that appears in the display or stands for mode's
Bold is used to refer to menu options and names of keys

1.2 Design

This treadmill called **M 100**, is build-up in a modular way.

1.3 Finally

You have made a wise choice in selecting the **M 100**. We are confident that your unit will continue to give satisfaction over many years of use. Nevertheless, if you have any queries or suggestions, please contact your authorized distributor.

2 Safety

2.1 Preliminary notes

It is important that you read these operating instructions carefully before using the **M 100**. Please make sure that these instructions are available to all personnel who operate the **M 100**.

Pay attention to the following before using the **M 100**:

1. Keep yourself informed of the contra-indications (see chapter 3).
2. The apparatus may not be used in close proximity (e.g. less than 2 meters) to short-wave equipment.
3. Never leave the patient unattended during treatment.
4. The apparatus may not be used in so-called "wet rooms" (hydrotherapy rooms).

The manufacturer cannot be held responsible for the results of using this apparatus for any purposes other than described in these operating instructions.

2.2 Safety belt

The safety belt has to be worn during running on the machine. The **M 100** will stop automatically as soon as belt contact is interrupted from the magnetic contact of the operating panel.

2.3 Emergency stop

The machine will stop automatically when the emergency stop has been pushed. The program can be started again after dislocation of this button by pulling it and pushing start.

2.4 Access ramp

An (optional) access ramp is available for wheelchair patients.

2.5 Product liability

A law on Product Liability has become effective in many countries. This Product Liability law implies, amongst other things, that once a period of 7 years has elapsed after a product has been brought into circulation, the manufacturer can no longer be held responsible for possible shortages of the product.

3 *Indications and contra indications*

3.1 Indications for cardiovascular training

Cardiovascular training is indicated if the following goals are set :

- Increasing/maintaining muscle strength.
- Increasing/maintaining stamina.
- Increasing/maintaining mobility/flexibility.
- Improving co-ordination.

3.2 Absolute contra-indications for cardiovascular training

- recent cardiac infarction, suspected infarction, pronounced resting stenocardia;
- pronounced resting insufficiency;
- serious dysrhythmia and asequence;
- pronounced aortic stenosis;
- active tromboflebitis, recent embolism;
- malignant hypertonia;
- myocarditis, endocarditis, pericarditis;
- major aneurysm of the heart or large vessels;
- acute infectious disorders, acute illnesses.

3.3 Relative contra-indications for cardiovascular training

- serious exercise coronary insufficiency;
- stimulus and conduction disorders;
- metabolic derangement;
- moderately severe aortic stenosis;
- hypertonic regulatory disorders;
- hypertrophia;
- respiratory insufficiency;
- minor aneurysm of the heart or large vessels.

4 *Installation*

4.1 Connection

- Mains supply connections must comply with the national requirements regarding medical rooms.
- Prior to connection of this apparatus to the mains supply, check that the voltage and frequency stated on the type plate correspond with the available mains supply.

4.2 Connection mains cable

- Connect the supplied mains cable to the connection mains cable. (figure 2)
- Connect the mains cable to a grounded safety socket.

4.3 Switching on and self test

- Switch on the unit using the On/Off button. (figure 2)
- Immediately after switching on, the unit carries out a self test.

Check that:

- All LED's light up briefly.
- At the end of the test a beep can be heard.

Contact your supplier if this is not the case.

4.4 Checking walking surface

Before daily use of the M, the belt has to be inspected on wear. By use of different services, it is possible that the belt has to be aligned. See for alignment the chapter 7.2 (Alignment of the belt)

4.5 System adjustments (figure 3)

Changing units.

As the M is delivered, all values are given in SI-units (metrical system). Switching units is realised from the stand-by situation by pressing:

- Press **Manual training** [1] and **Stop** [4] at the same time for a minimum of 10 seconds.
- By pressing **Start** [3] the next values are visible.

Total Hr=	Total hours the M has performed
Service	= Total kilometers the M has performed
Belt count	= Total amount of kilometers the belt has performed
Hi speed=	Maximum speed of the belt.
Duration	= Maximum time during 1 training (99 =infinite)
Brightness electronics	= brightness of the display. Recommended is 4 or 5. because of the workload of the
Display mile	= 0 =kilometers/ kilogram 1 =miles/pounds

All values can be changed with + and - under [8]

4.6 Installation

- Do not install the unit in a location near to a heat source such as a radiator.
- Avoid exposure to direct sunlight, rain, excessive dust, dampness, mechanical vibrations and shocks.
- Cordless telephones should not be used in close proximity of the unit.
- This unit should not be used in so-called "wet-rooms" (hydrotherapy rooms).
- The unit has to be installed in such way that liquid cannot enter.
- A proper air circulation is imperative to prevent overheating, so do not leave clothing etc. on the motor compartment cover.

If during training no heart-rate is received, the **indicator** [10] shows no Hr. As soon as the heart-rate is received, it will be displayed again.

After finishing a training, it's possible to read out the amount of performed kilo calories.

During execution of the program the **display** [7] will alternately indicate the values:

- Time
- Minimum heart rate: L (Low pulse)
- Time
- Maximum heart rate: H (High pulse)

After finishing the program, the display will show the amount of performed kilo calories.

Each protocol training can be stored until 9 protocols per mode. These protocols can be changed on every moment.

A quick start of a protocol is made as described underneath:

- Choose type of training with one of the **buttons** [1]. The display will show program with a number.
 - With **+** and **-** [8] it's possible to go to a next program number.

To go to another step in a program: push the **program with time and heart-rate-button**. Confirm with **Start** [4].

After entering start speed and weight, push **Start** again. The belt will start to rotate.

5.3.5 Quick start

(all models)

The simplest way to start the **■** is this quick start mode. No parameters have to be filled in and the **■** is able to start immediately by pressing **Start** [3]. This type of training does not show the amount of performed kilo calories.

5.3.6 Heart-rate

By means of the **Polar** belt (necessary for cardio training and training protocols with time and pulse) the actual heart-rate can be seen on the actual **heart-rate display** [10]. If no **Polar** belt is being worn, no heart-rate is visible.

5.3.7 Stopping a program

During training the **■** can be stopped by pressing **Stop** [4]. When this button is pressed ones, the **■** will stop slowly and turn into pause situation. Pressing **Stop** again will turn the **■** into the stand-by mode while the last parameters will be shown.

6 Maintenance

6.1 Technical maintenance

We recommend to have the unit checked annually. This may be done by your supplier or by another agency, authorized by the manufacturer. It is also recommended that a record of the service history is kept for all activities relating to service and maintenance. In some countries this is even obligatory.

Maintenance and all repairs should only be carried out by an authorized agency. The manufacturer will not be held responsible for the results of maintenance or repairs by unauthorized persons.

The check up and/or technical maintenance must be carried out conform the procedure described in the service manual of the unit.

Opening of the equipment by unauthorized agencies is not allowed and will terminate any claim to warranty.

6.2 Alignment of the belt

If the treadmill has to be used on a floor that is not flat, the followi

1. Adjusting the speed of the mill to 5 km/h.
2. If the mill belt runs towards the left side, as viewed from the rear of the **M11**, adjust the left belt adjustment bolt at the rear end of the belt (using a 6mm socket head screw wrench) a quarter turn clockwise. (see figure 4). The belt will now move slowly right.

If the mill belt runs towards the right side, as viewed from the rear of the **M11**, adjust the left belt adjustment bolt at the rear end of the belt (using a 6mm socket head screw wrench) a quarter turn anti-clockwise. (see figure 4, page 72). The belt will now move slowly left.

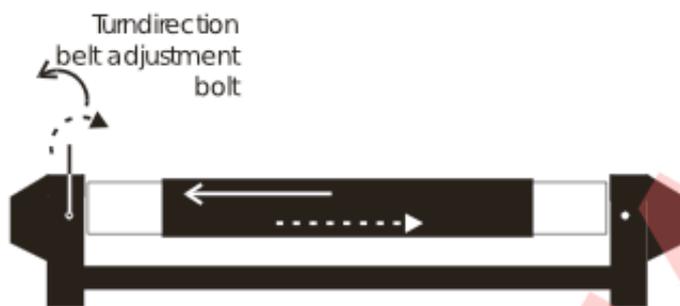
ATTENTION!

The belt reacts very slowly to the turning of the tension bolt. Therefore let the treadmill run for 2 min.

If this adjustment is not enough, repeat the action mentioned under number 2 again.

ATTENTION!

ONLY use the left adjustment bolts since the other bolts will influence the belt tension. This could degrade the performance of the **M11**.

Backside / rear view

Figure 4 – adjustment of the belt

The figure 4 shows the direction the belt adjustment bolt should be turned in order to move the belt in the shown direction.

6.3 Maintenance (by the user)

Before cleaning and other maintenance of the treadmill, first switch the off the **M11** and disconnect it from the mains. The **M11** can be cleaned with a damp cloth and a liquid (non-abrasive) household cleaning product. The belt can be cleaned with a sponge, with little lukewarm water. Allow the belt to dry thoroughly after cleaning before re-using the treadmill.

6.4 End of life **M11**

Your **M11** contains materials which can be recycled and/or are noxious for the environment. At the end of the duration of life, specialized concerns can take apart the **M11** and sort out the noxious materials and materials for recycling. By doing so you contribute to a better environment.

Please ensure that you are well informed of the local rules and regulations regarding to the removal of equipment and accessories.

7 Trouble shooting

7.1 Display does not light up

- Check that the mains plug is correctly inserted in the wall-socket.
- Check the mains supply to the wall-socket.
- Check the fuses in the mains output and replace if necessary.
- Check the brightness adjustment of the LED-display

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