Chapter 6 System Parameters Setting

The system parameters of this instrument can be set in the menu.. The displayed parameters in the menu are the factory default value, and the users can set the parameters which need to be changed before electrocardiogram recording.

6.1 Main menu

After starting the instrument , press the  button to enter into the Menu setting interface , users can press   button to move the Move the selection box to each option as shown below:

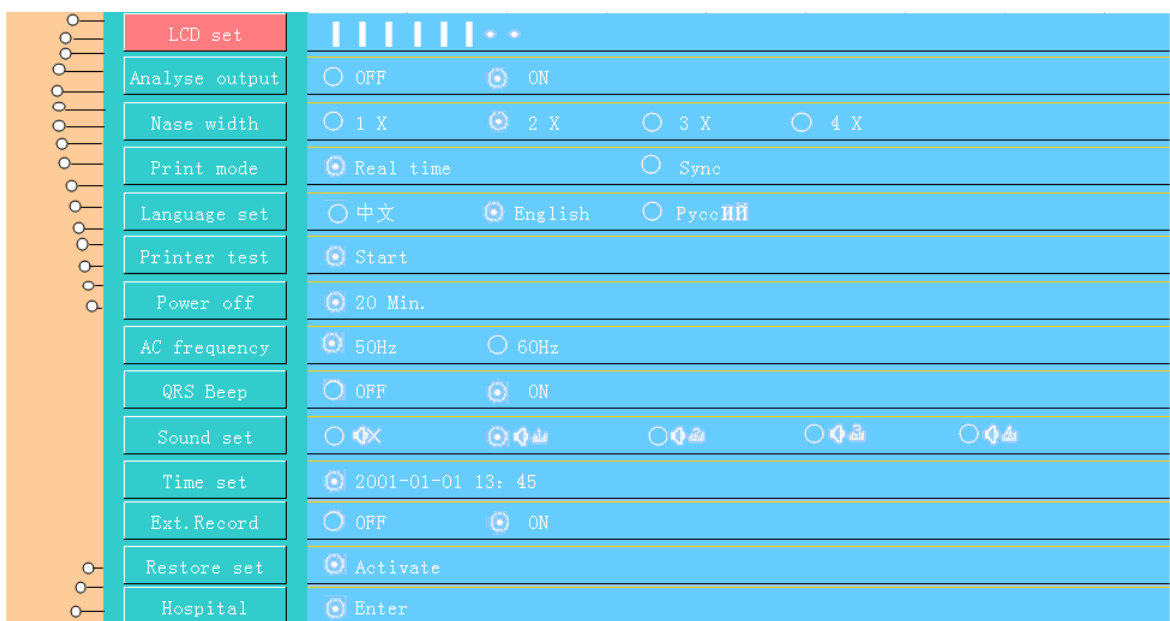
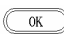
Figure 6-1 Main menu interface

In the Menu setting interface, the pink background is the current selected sub-menu, press  button to enter the lower menu, press   button to move the selection box for the other option.

Parameters list :

Table 6-1 Parameters list of main menu

Parameters	Setting Value	Defaulting Value	Comment
LCD set	··	··	Adjust the LCD contrast
Analyse output	OFF , ON	ON	Automatic analysis of report output options
Base width	1x、 2x、 3x、 4x	2x	Select the width of the waveform baseline

Print mode	Real time , sync	Real time	Select data collection print mode
Language set	English,	English	Setting the display language
Printer test	Start	Start	Print a test triangle
Power off	0min、 5 min、 10 min....60 min	20 min	Set Auto shutdown time
AC frequency	50Hz、 60Hz	50Hz	Network Power Frequency Selection
QRS Beep	OFF , ON	OFF	Select Heart Rate Detection tone switch
Sound set			Select voice tone and switch
Time set	XXXX-XX-XX XX: XX	The current time	Year - month - day hour: minute format input
Ext. Record	OFF , ON	OFF	Select the external input signal record
Restore set	Activate	Activate	Restore the state of factory setting
Hospital	Enter	Enter	Input hospital name information

■ LCD set

Move the selection box to “LCD set” , press  button and then press   button , users can adjust the LCD contrast.


■ Print mode

Real-time print mode is based on changes in the human body in real time as a ECG recording of printing method; The synchronization is the standard twelve lead ECG data is synchronized, it means that each lead waveform situation at the same period of time . Data for each lead is the same starting time. Users can easily observe the data of each lead at the same time period.

Note :

Synchronous printing mode is only effective in the automatic mode.

■ Printer test

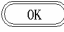Move the selection box to “Printer set” , Press  button , then the instrument can print the triangle wave as shown below, Users can observe the triangular wave whether there is a breakpoint, bending or wave phenomena, and then test the print head is in good condition or not.

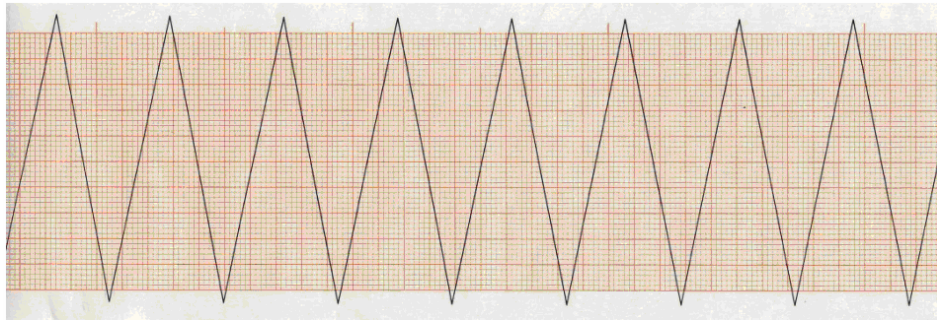
Figure 6-2 Printer testing page of the information on the recorder

- **AC frequency**

This setting needs to match the frequency and use the AC, if you set it improper; it will affect the mapping results and analysis.

- **External Record**

External input is set as "On ", the machine enter into an external input screen, the screen display as shown below:

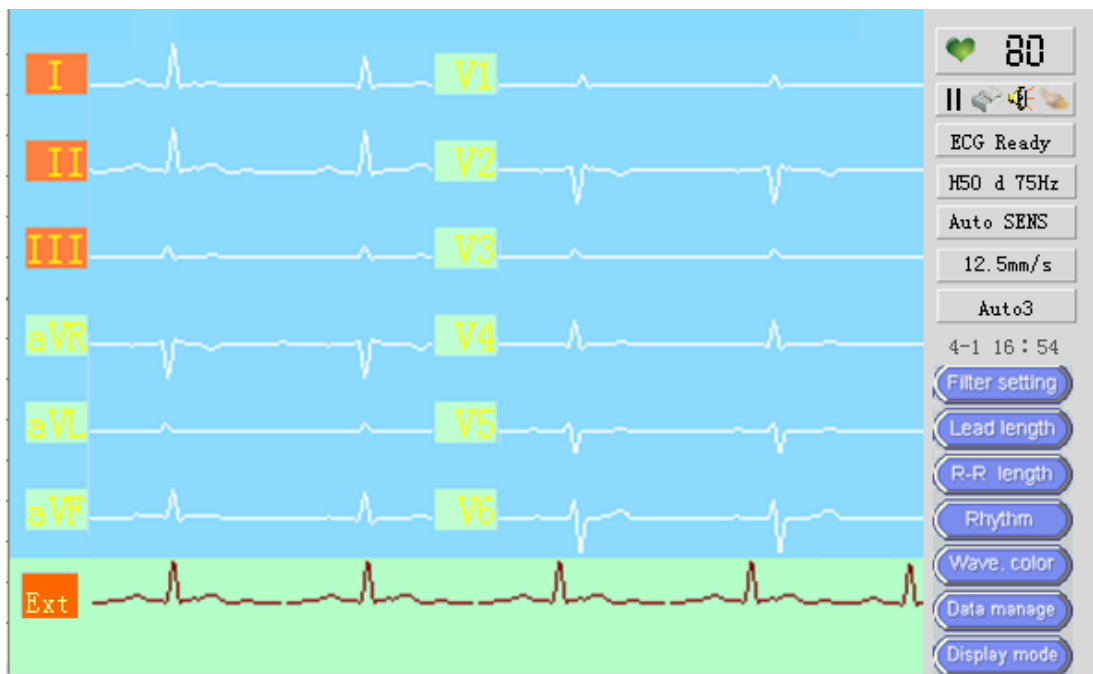
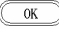
Figure 6-3 External Record status

Press  button (Start/Stop record button) to print the external input waveform.

- **Hospital information**

Move the selection box to "Hospital" , press  button to enter the hospital edit interface:

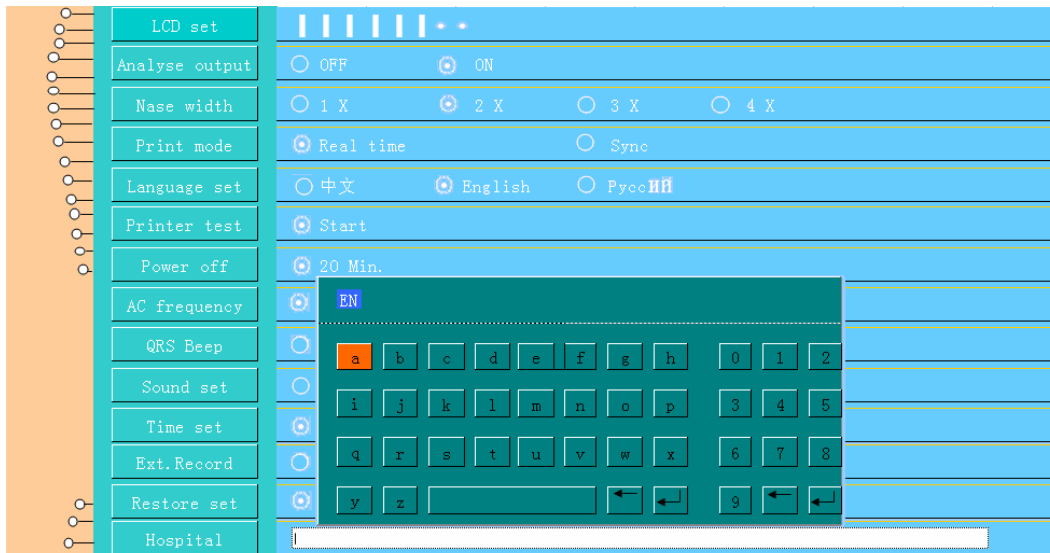
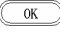
Figure 6-4 Hospital edit interface

Users can select the CH and EN to input, detailed operation please see Chapter 7 “ Name inputting”.

6.2 Parameter setting

In the main interface, press the  button , and then press  button , users can move the selection box to set each parameter, as shown below:

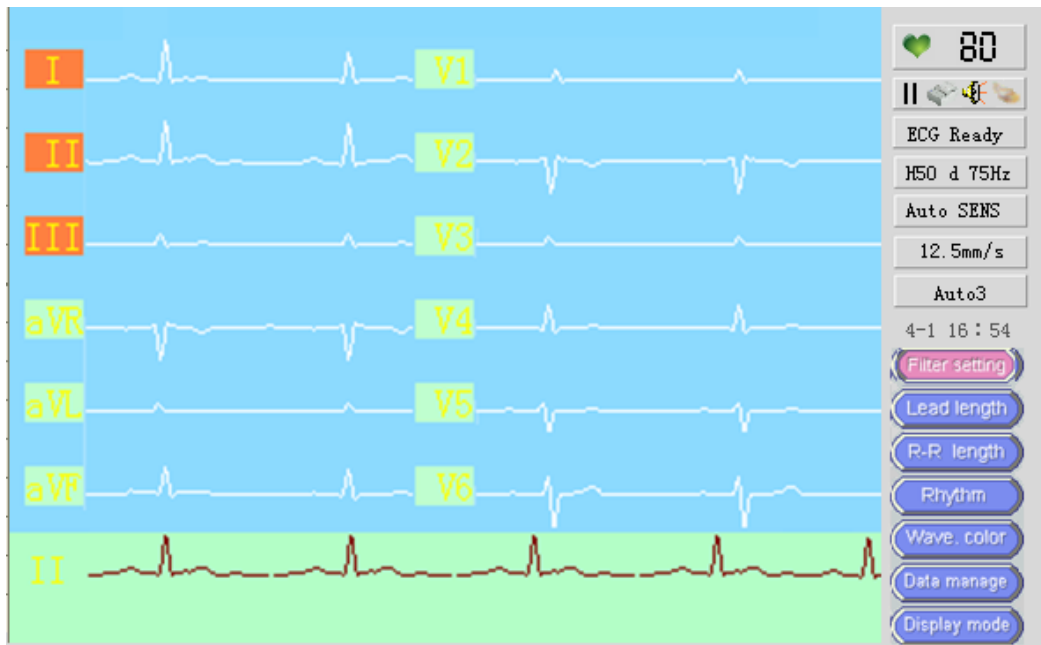
Figure 6-5 Parameter setting interface

Table 6-2 Parameter setting


Parameters	Setting Value	Defaulting Value	Comment

Filter setting	0.5-150Hz , H50 d , H50 d 75Hz , H50 d 35Hz	H50 d 75Hz ,	Filter setting
Lead length	3s—12s	3s	The length of a single channel recording time
R-R length	30s—180s	60s	Selection R-R duration lead length
Rhythm	□ , □ , □.....V4,V5,V6	□	Select the rhythm lead
Wave.color	White, black, red, green, blue, yellow, pink, blue, orange, purple	white	Select the display color of lead
Display mode	3 , 6 , 12	12	Select the display mode of lead

Chapter 7 Patient Information and Data Administration

This chapter introduces how to input and display the patient information, as well as operation procedures of storing and transmitting of ECG data.

7.1 Patient Information Inputting

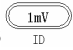In the main interface, press  button, the patient information interface pops-up, after setting it , press button to return to the main interface.



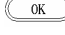
Figure 7-1 Patient information inputting interface

The screen displays the patient name, age, gender and ID number four setting items, users can input the information according to the actual situation of the patient.

■ Name inputting

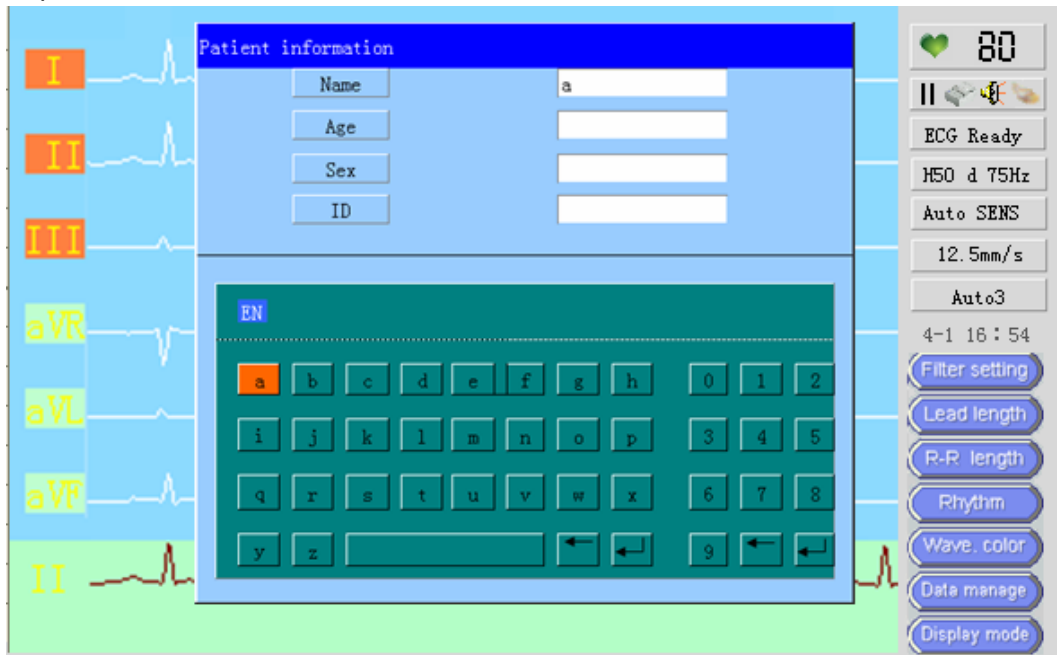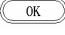Moving the selection box to select "Name", and then press the  button or the   button to move the selection box to the "EN" or "CH", users can press the  button again to switch the input mode.

Letters input

Select the input mode "EN", in the letter keyboard enter the corresponding letter, and press the  button to return to achieve the information inputting.

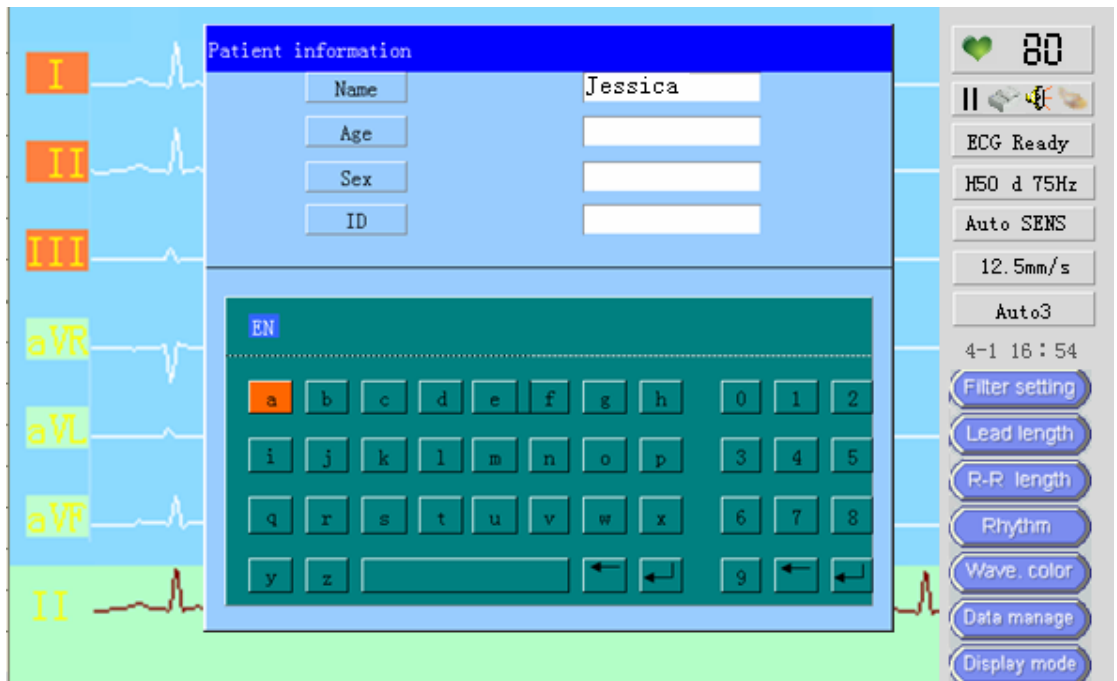
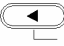
Figure 7-2 Letters input interface


■ Age inputting

when users is inputting the Age , the system will automatically to stay the selection box in the numeric keypad, in the numeric keypad, users can move the selection box to select the desired number of input, and then press  button to input the information.

■ Sex selection

The default is "None", "None" means that in the report the gender column is blank, it is filled by the doctor, press the  button to select "Male", "Female" or "None ", and then press  button to return.

■ ID

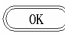The machine automatically to give a unique ID number for collected ECG data ,and named the ID number to store ECG data; in this option, press the  button, the instrument will prompt weather to clear ID(as shown below).

Because the storing is named by the ID number, if you choose "YES", it will result in duplication of storage, at last resulting in data management confusion.

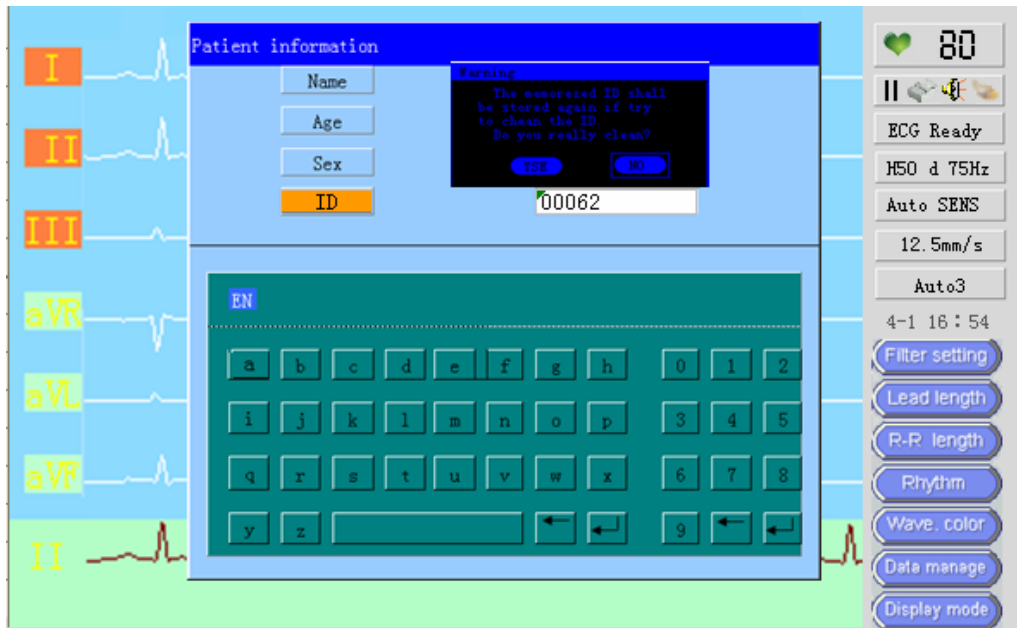
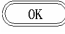
Figure 7-3 ID Input Warning

7.2 Data Manage

This instrument provides the storage medium internal memory for storing ECG ,Analysis and measurement recording ECG data can be stored in the storage medium.

In the main interface, press  button ,then press   button to move the selection box to "Data manage", after selecting, press , the input interface will display: :

Saved: 13 Total size: 2000 [Page:1]

NO.	ID	Name	Date	NO.	ID	Name	Date
1	00000	None	2011-3-31 8:52				
2	00001	None	2011-3-31 8:59				
3	00002	None	2011-3-31 9:12				
4	00003	None	2011-3-31 9:22				
5	00004	None	2011-3-31 9:52				
6	00005	None	2011-3-31 9:59				
7	00006	None	2011-3-31 10:22				
8	00007	None	2011-3-31 11:52				
9	00008	None	2011-3-31 12:52				
10	00009	None	2011-3-31 12:22				
11	00010	None	2011-3-31 12:32				
12	00011	None	2011-3-31 13:52				
13	00012	None	2011-3-31 15:32				

Cue bbutton[<-][>-] =[previous] [next]

Figure 7-4 ECG data stored in the storage medium

When the instrument complete an ECG recording, ECG analysis results, and patient electrocardiogram data constitute a file saved in the default storage medium for Data query and management in the future.

■ Internal Memory

The instrument's internal memory can store up to about 2,000 copies of ECG data (the actual situation according to the liquid crystal display). When the data storage is full, the new ECG data will automatically replace the first ECG data. The current state of storage media is displaying on the screen, Store file and the patient's ID number in sequence to be storied, users can query and playback the waveform.

In the data storage screen, move the selection box to select the inspection ECG data, and then press the button to pop-up such dialog box as shown below:

NO.	ID	Name	Date	NO.	ID	Name	Date
1	00000	None	2011-3-31 8:52				
2	00001	None	2011-3-31 8:59				
3	00002	None	2011-3-31 9:12				
4	00003	None	2011-3-31 9:2				
5	00004	None	2011-3-31 9:5				
6	00005	None	2011-3-31 9:5				
7	00006	None	2011-3-31 10:0				
8	00007	None	2011-3-31 11:0				
9	00008	None	2011-3-31 12:0				
10	00009	None	2011-3-31 12:0				
11	00010	None	2011-3-31 12:0				
12	00011	None	2011-3-31 13:52				
13	00012	None	2011-3-31 15:32				

Saved: 13 Total size: 2000 [Page:1]
 Cue bbutton[<-][>-] =[previous] [next]

Figure 7-5 Data management operations

There are four options for user using: "open", "Data retrieval", "upload" and "Format", users can carry out the necessary processing and operations for ECG data.

■ Open file

Select "Open" option to playback the waveform and data access, LCD display the following "data playback" picture:

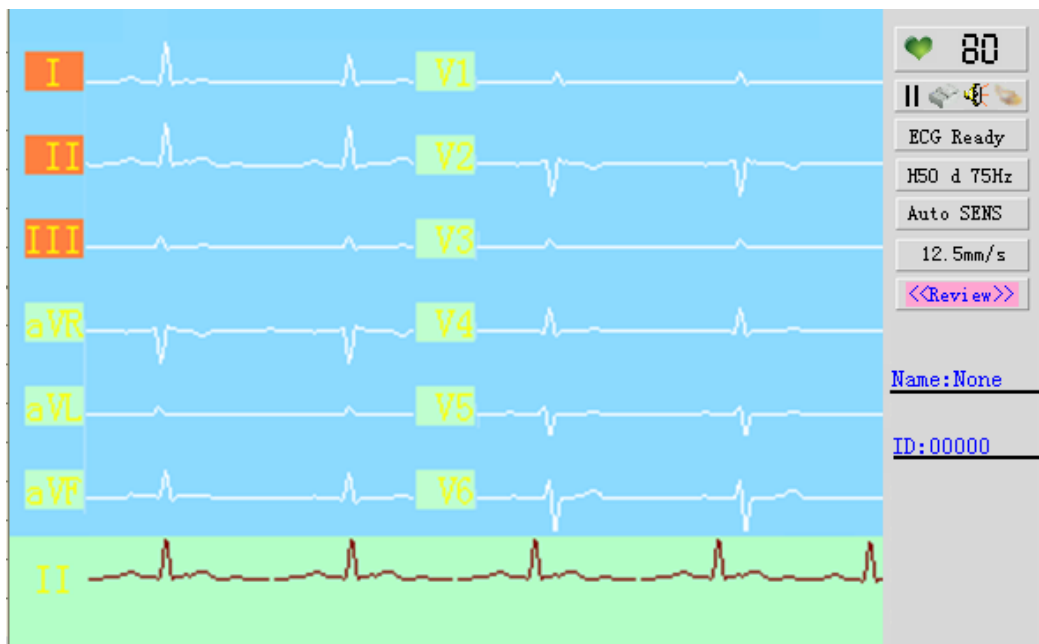
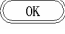
Figure 7-6 Data playback interface

This picture include 12-channel ECG waveform, the patient name, ID number , select the paper speed and press the button to print this ECG data.

Print ECG data in the "Wave Playback" mode, the instrument will automatically to print it as "Auto 3" mode, after printing the waveform, it will automatically to print the stored analysis results and the patient name ,ID and time.

■ Data retrieval

Select "Data retrieval" option , and press  button :

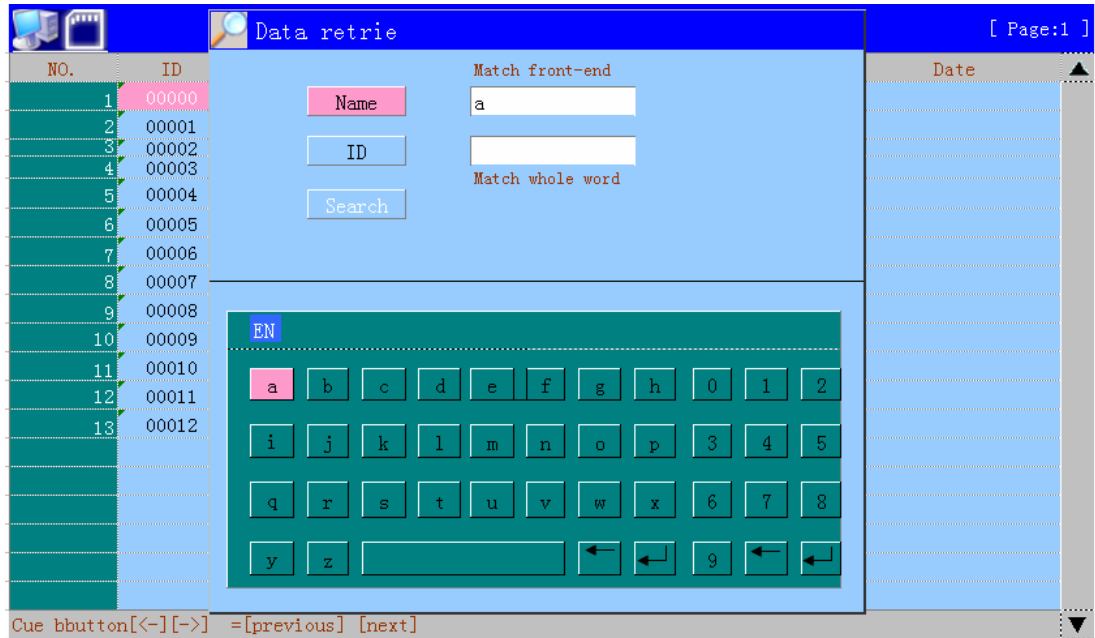
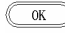
Figure 7-7 Data retrieval interface

There is "Name" and "ID" option in this dialog, in this dialog, users can retrieve the required ECG data.

■ Upload file

Select "Upload" option , and press  button :


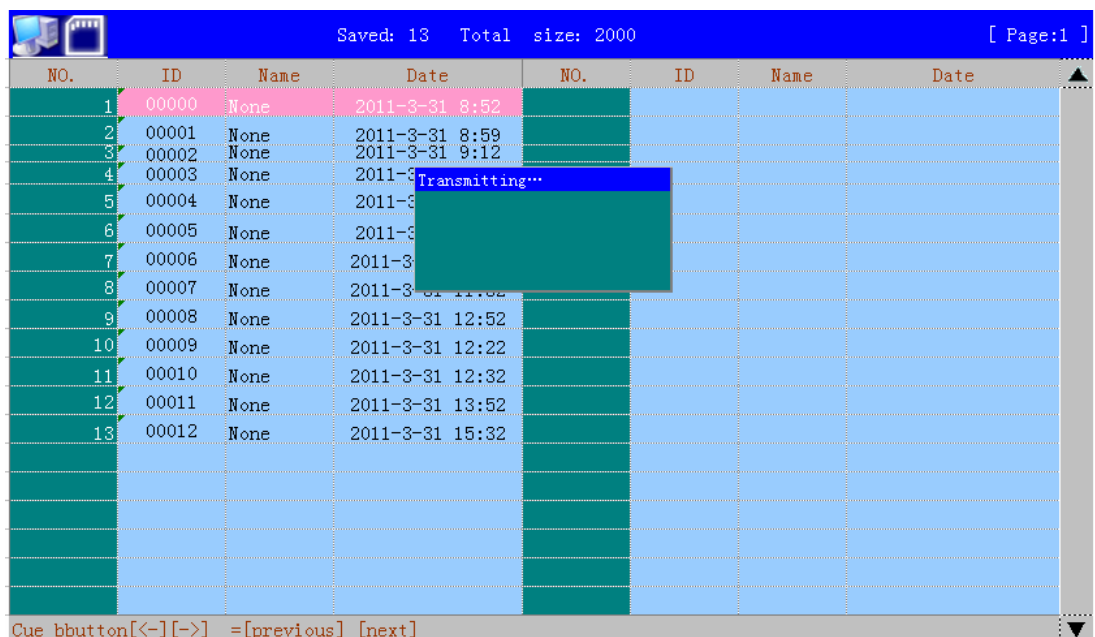
Figure 7-8 Upload file interface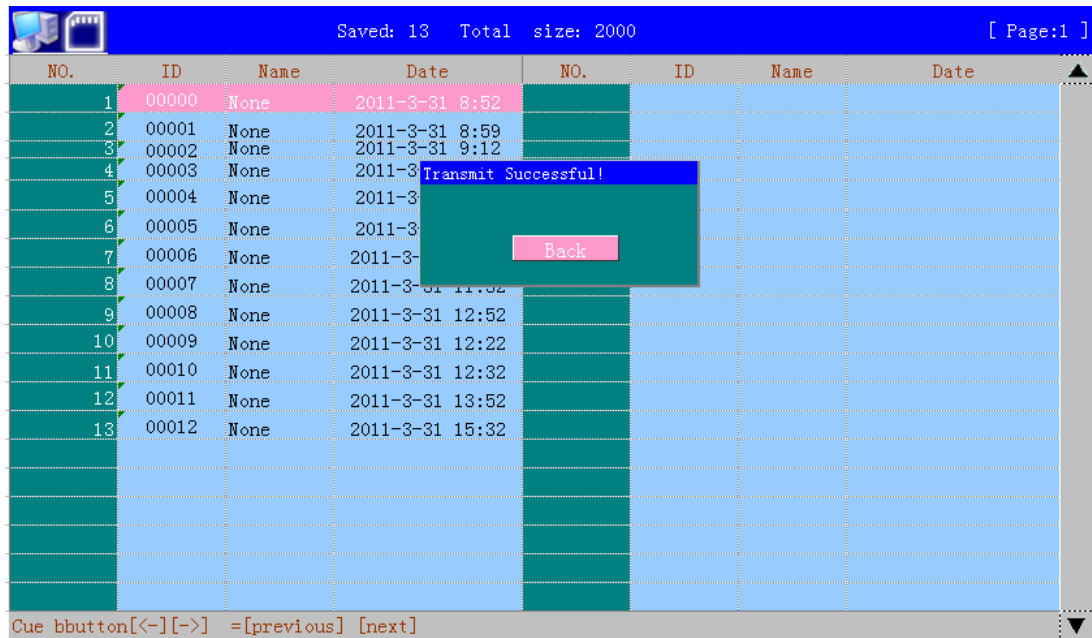

NO.	ID	Name	Date	NO.	ID	Name	Date
1	00000	None	2011-3-31 8:52				
2	00001	None	2011-3-31 8:59				
3	00002	None	2011-3-31 9:12				
4	00003	None	2011-3-31 9:22				
5	00004	None	2011-3-31 9:52				
6	00005	None	2011-3-31 9:52				
7	00006	None	2011-3-31 11:52				
8	00007	None	2011-3-31 11:52				
9	00008	None	2011-3-31 12:52				
10	00009	None	2011-3-31 12:22				
11	00010	None	2011-3-31 12:32				
12	00011	None	2011-3-31 13:52				
13	00012	None	2011-3-31 15:32				

Cue bbutton[<-][>-] =[previous] [next]


Figure 7-9 Upload file Successful interface

Note:

The data transfer function must be related to the user has the hardware, software support, Specifically refer to the instructions in this section," Management of Communication between the Product and Computer

- **Format**

Select "Format" option , and press button :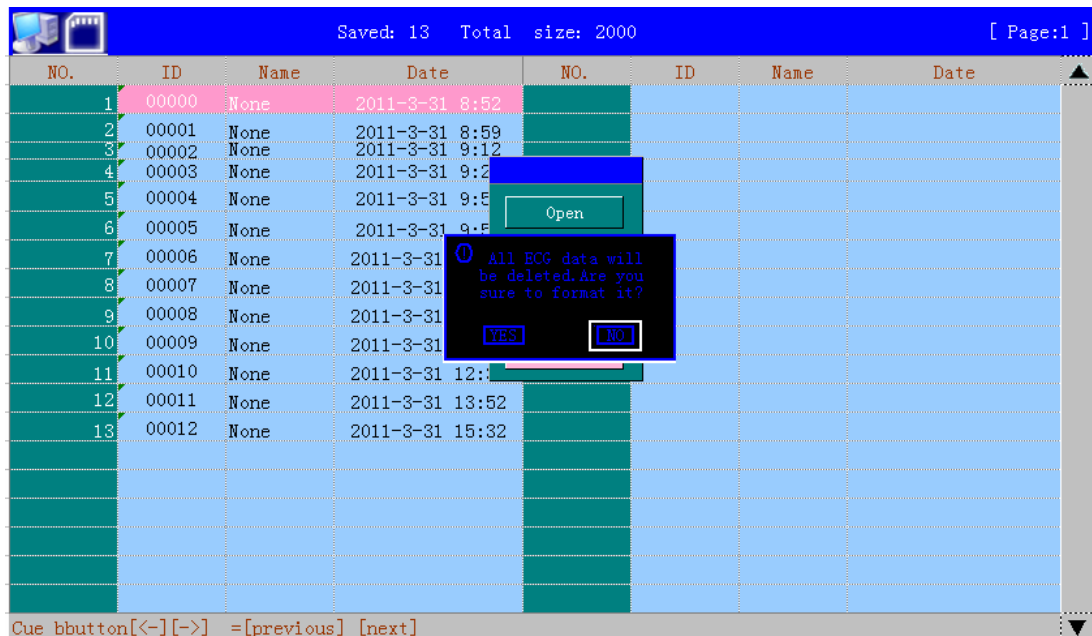

NO.	ID	Name	Date	NO.	ID	Name	Date
1	00000	None	2011-3-31 8:52				
2	00001	None	2011-3-31 8:59				
3	00002	None	2011-3-31 9:12				
4	00003	None	2011-3-31 9:22				
5	00004	None	2011-3-31 9:52				
6	00005	None	2011-3-31 9:52				
7	00006	None	2011-3-31 11:52				
8	00007	None	2011-3-31 11:52				
9	00008	None	2011-3-31 12:52				
10	00009	None	2011-3-31 12:22				
11	00010	None	2011-3-31 12:32				
12	00011	None	2011-3-31 13:52				
13	00012	None	2011-3-31 15:32				

Cue bbutton[<-][>-] =[previous] [next]

Figure 7-10 Warning of Format

The system prompts whether to format the memory, If format memory, the memory of all the corresponding records will be deleted, and can not be restored, please be sure!

7.3 Management of Communication between the Product and Computer

The product can conduct data transmission to computer through RS232 port. Before using this function, make sure that the product has been reliably connected to the computer through RS232 cable, and that the ECG management software designed by the computer has been installed in the computer. The connection between the product and the computer is as follows:

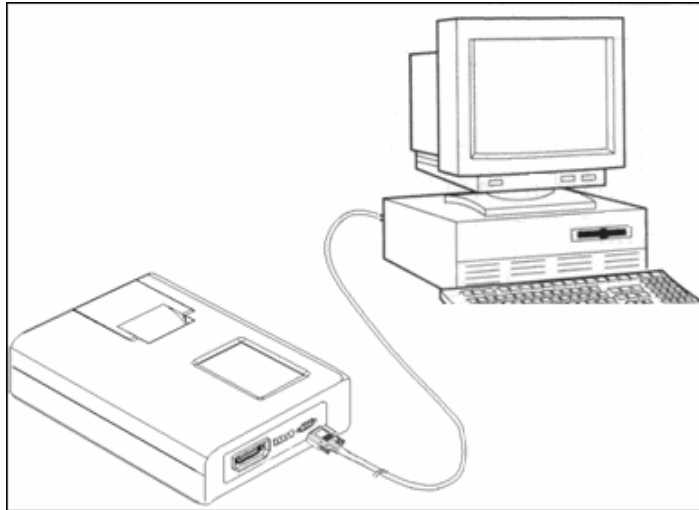
Figure 7-11 connection between the product and the computer

When the product is connected to the computer, select “Transmission” in the function menu of the product function and press button to realize data transmission.

Note:

- 1、RS232 connection cable should be the cable provided by the company along with the ECG management software or with the same specifications as it.
- 2、For detailed functions and operations of the ECG management software, please refer to the manual of ECG management software.
- 3、The ECG management software is an optional product, which can be purchased by contacting business personnel of the computer in case of need.

Chapter 8 Servicing and Maintenance

8.1 Matters needing attention about the repair and daily maintenance of the product

- a . There is no part inside of the product that requires the user to conduct maintenance, the user's maintenance work normally involves the appearance cleaning of the product and checking of

the effectiveness of controlling buttons and integrity of the outer insulating protection of the leads, thus the user should not dismantle the machine without approval;

- b . When abnormal conditions of the product appear, mark the part with problem and conduct repair immediately, and do not use products with malfunctions. As there may be high voltage inside of the product that would be harmful to human, the repair of the product must be conducted by specialists with qualification, no other person shall open the body case of the product to conduct repair. The technical materials such as electrical schematic diagram and parts list that are needed for the repair of the product can only be provided to the repairing personnel with qualification;
- c . Repair of product must be conducted according to the original design, and no model alteration is allowed. After repair, the product must go through necessary safety examination and measurements before it can be used;
- d . The product and all its accessories must be maintained regularly, with a time interval no longer than half a year.
- e . The product is the force measuring instruments, users should statutory qualifications to the local institutions to apply the measurement test, test period is not less than one year.

8.2 Maintenance and Replacement of the Built-in Battery

The built-in battery of the product is rechargeable battery which is installed inside of the product. As the service life of rechargeable battery is related to the times of charging and discharging, it is recommended to use AC power supply when the conditions permit to prolong the service life of the rechargeable battery. The electric capacity of the built-in rechargeable battery is displayed on the monitor of the product when using the battery to work, so the user can know whether the battery needs recharge through observing the electric capacity of the battery. When the battery is fully charged but the working time is less than 30 minutes, or the battery cannot be recharged, the built-in battery may have lost its effectiveness and should be replaced. As the body case of the product should be dismantled to replace the built-in battery, the replacement of the built-in battery should be conducted by professional maintenance personnel, and the user should not replace it on himself/herself in order to avoid electric shock. Meanwhile, the rechargeable battery with the same specifications must be used to replace the built-in battery to prevent accidents from happening. The detailed steps to charge the battery, display the electric capacity and protect the battery are explained as follows:

8.8.1 Recharging the Battery

There are battery charging and protecting circuits designed inside of the product. When the AC power cord is connected, the battery can be charged. When the product is not use for a long time, at least charge and discharge the battery every three months. When charging the battery, turn the product into standby status, and open the power switch at the back the

product. During charging process, the charging indicator lamp on the front face of the product will blink, and when the charging is completed the indicator lamp will illuminate constantly.

8.8.2 Capacity Display of the Battery

When using battery to supply electricity, there would be a icon that represents the capacity of the battery in the monitor after the machine is turned on, as shown in the figure below. There are four shapes of the icon which indicate different capacities which are explained below:

: Battery capacity is full;

: Battery capacity is sufficient;

: Battery capacity is not sufficient, needs recharge;

: Battery capacity is empty, needs immediate recharge;

8.3 Recording paper

For ensuring the quality of the ECG waveform tracing, the high speed heat sensitive recording paper designated or provided by our company should be used. Using the undesignated recording paper may cause problems such as the service life of heat sensitive printing head falling , the unclear waveform recording and paper jam and should be given particular attention.

- (1) Do not use the recording paper coated with wax , and the paper in the color of ray or black, or else the wax may adhere to heater of the printing head and lead to the printing head failure or damage.
- (2) High temperature, moisture and sunlight will cause the color change of the recording paper, thus keep it in the dry and cool environment .
- (3) Do not place the recording paper under the fluorescent light for long term or else the recording quality may be affected.
- (4) Do not place the recording paper with the PVC plastic material or else it may cause the color change of the recording paper.
- (5) When the recording paper is placed in cascade, the recorded waveform may be printed from one piece to another one each other.
- (6) Particular attention should be given to the size of the recording paper. The recording paper of which size is not in compliance with the requirement may cause the damage of the heat sensitive printing head or the silicon rubber roll.

8.4 Instrument maintenance after using

The following matters should be given particular attention after using the ECG instrument:

- (1) Press the ON/OFF button on the control panel to set the instrument in the stand-by status.
- (2) Hold the plug to pull off the power supply cable and the lead lines instead of holding the cable.
- (3) Clean up the instrument and accessories, then place the dust cover on it.

- (4) Keep the instrument in the dry and cool environment. Prevent it from intense shock when moving it.
- (5) Do not soak this instrument into the detergent when cleaning it. Please cut off the power supply first when cleaning the outer shell of the instrument . Please clean the instrument with the neutral detergent instead of using the detergent containing alcohol or using the sterilizer.

8.5 Inspection and maintenance of lead lines and electrodes

- 1 The continuity condition can be tested by the multimeter. Please inspect the contacting condition of each lead line based on the following table . The electric resistance between the electrode end and the lead plug pin for each lead line should be less than 10Ω. Please inspect the continuity condition of the lead lines regularly, since the damage of any one line among them will cause the false waveform of corresponding lead or the entire lead in the electrocardiogram. The lead lines can be cleaned with the water and soap as well as sterilized with 75% alcohol (do not soak the lead lines into the liquid for sterilization).

Note: The electric resistance of the lead line with the protective function of anti-defibrillation is 10kΩ approximately.

Table 8-1 Symbol for connector plug and Position for plug and pin

Symbol for connector plug	R	L	F	RF	C1	C2	C3	C4	C5	C6
Position for plug and pin	9	10	11	14	12	1	2	3	4	5

2. Bending the lead line in the sharp angle or knotting will decrease the service life of the lead line, thus please perform the electrodes connection after tidying the lead line as possible .
3. The electrodes should be stored well. After using for long term, the oxidation color change of the electrodes may occur due to corrosion and so on , and the electrocardiogram may be affected. If this happens, please replace the electrodes in time.

8.6 Maintenance of silicon rubber roll

The silicon rubber roll should be kept in a stable, smooth and clean condition , or else the quality of the ECG recording may be affected . To clean off the stain on the silicon rubber roll, please wipe it along the long axis by a clean soft muslin with a little alcohol, while turning the silicon rubber roll along the transmission direction of recording paper to clean it entirely.

8.7 Cleaning of the heat sensitive printing head

The definition of the recorded waveform will be affected by the stain and dust on the surface of heat sensitive printing head. To clean the surface of the printing head, you can open the paper compartment cover after shutting down the instrument , then wipe the surface of printing head gently by the clean and soft muslin with a little alcohol . The residual contaminant on the printing head should be wetted with a little alcohol before removing with the soft muslin. Do not scrap the surface of printing head with the hard article or else the printing head will be damaged. Do not close the paper compartment cover before the alcohol is evaporated entirely .The printing head should be cleaned once per month when normally used.

8.8 Fuse replacement

If the AC power indicator light does not light when the AC power cable is connected and the power switch at the back of the instrument is on , as well as the instrument is unable to be started by pressing the ON button on the control panel or the battery status symbol as  is displayed after starting the instrument. If the AC power supply at the socket is confirmed to be in a normal condition, it is probable that the fuse has blown. At this time the AC power supply fuse should be replaced . The fuse is located in the internal portion of the instrument and should be replaced by the professional or the manufacturer.

- a . Cut off the AC power supply of the product;
- b . Turn the product up side down and insert a screwdriver into the tube fuse seat at the bottom of the product and turn the screwdriver until it is separate from the product;
- c . Take out the tube fuse in the tube fuse seat to check whether the fuse is melted down, if so, replace a tube fuse with the same specifications, otherwise check the other tube fuse;
- d . Install the new tube fuse into the fuse cap and align it to the position of the original tube fuse seat and use a screwdriver to fasten it clockwise;
- e . If the two tube fuses are not damaged after inspection, but the product cannot work with AC power supply, then the internal circuit may have malfunction, and the product should be sent to repairing organization with qualification to be repaired;
- f . When a new tube fuse is replaced and it is melted down when connecting power supply, then there may be other potential troubles in the internal circuit of the product, and the product should be sent to repairing organization with qualification to be repaired.

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